

Q Stat Safety Syringe

www.qstat.com.au

Instructions for Use

Do not use **3ml** and **5ml** Q stat syringes with needles longer than **1½ inches** (38mm)

Do not use **10ml** Q stat syringes with needles longer than **2 inches** (51mm)

Fluid may leak from needle after activation

Do not use with needle-less access ports

Do not use if packaging is damaged


Latex free

Single use device
Do not resterilize

Rx Only


1

Select the appropriate gauge and length of needle required and attach via the luer lock connection. Remove needle cover.

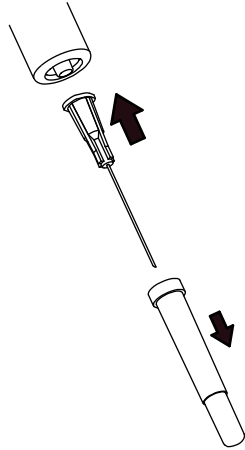


2

Prime the safety mechanism by depressing the plunger. You will feel a subtle click.

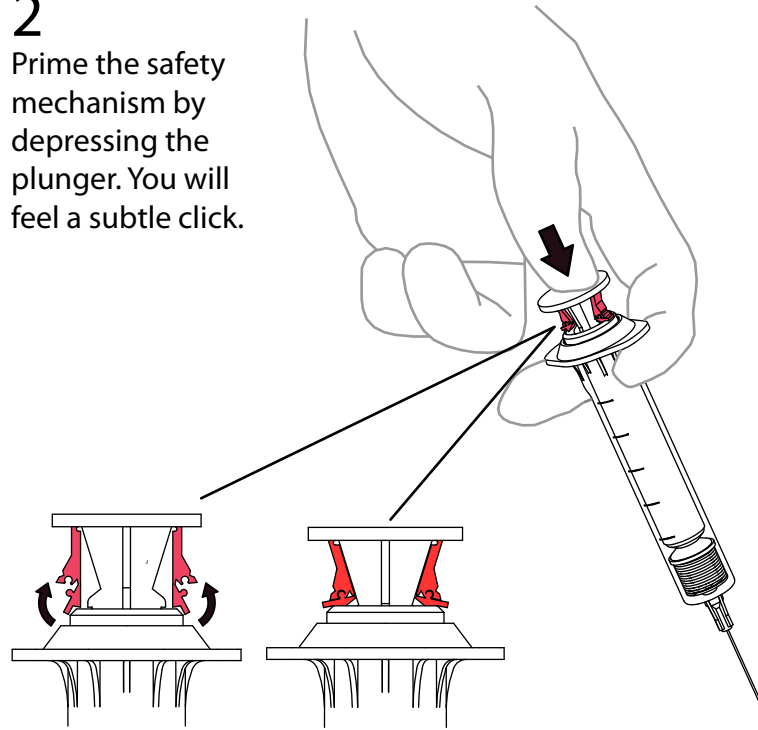

3


Draw up the medication in accordance with established protocol.

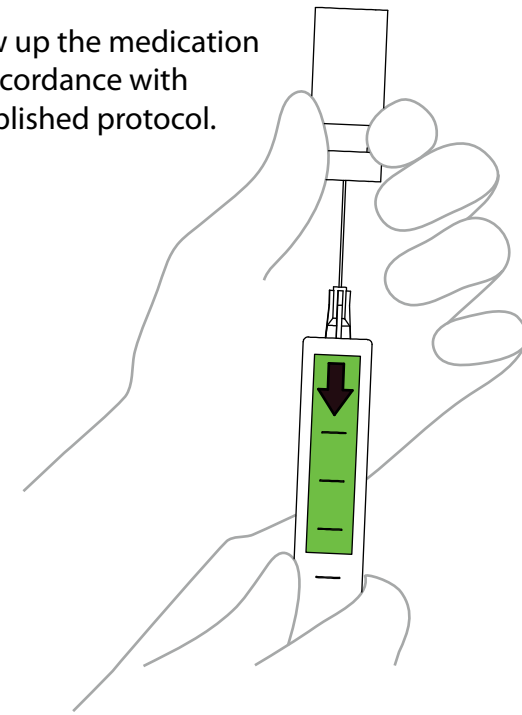

4

Administer medication in accordance with established protocols.


At the end of injection the safety retraction mechanism is automatically activated, no extra push is required.

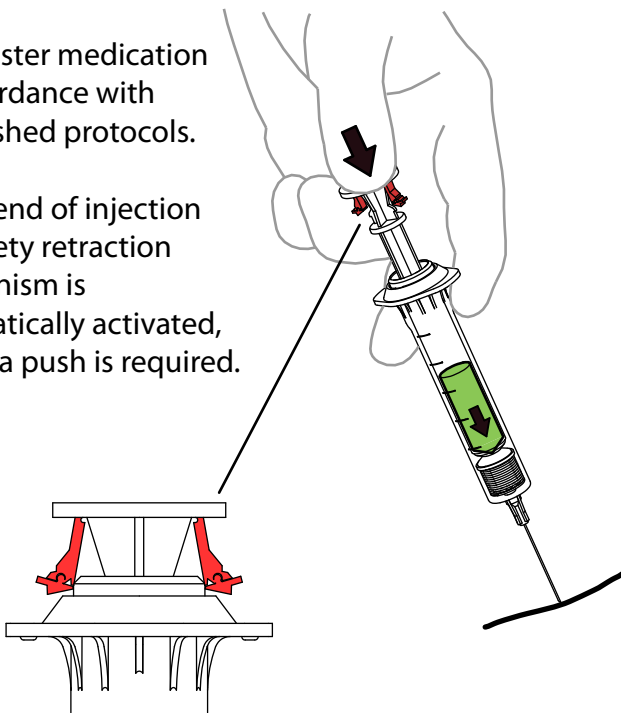

5


Needle retraction occurs when full dose is administered and pressure is slowly released from the plunger. Syringe is disabled when fully retracted.

This product allows needle retraction to be initiated inside the patient, or outside the patient, according to your institution's protocol.

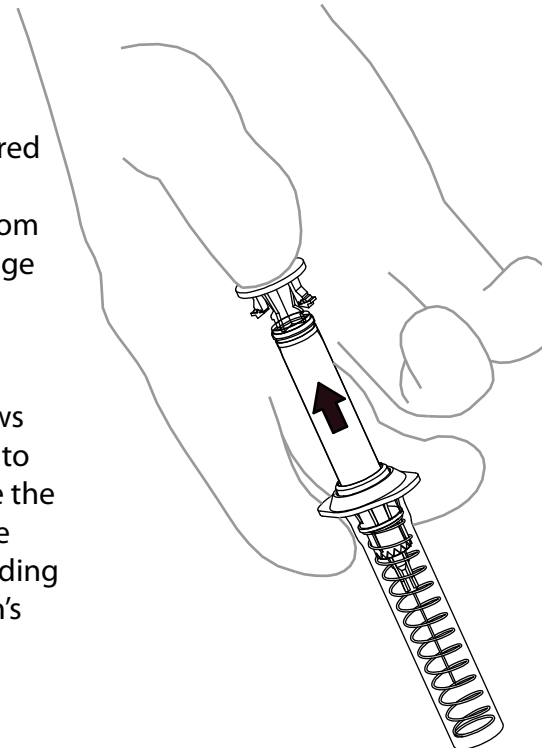

6

Discard immediately in nearest sharps container.

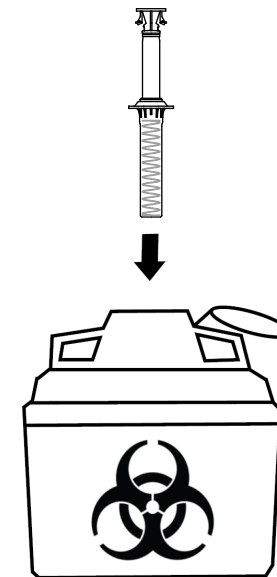

Caution: Reuse of syringes may cause cross infection

SUNNIST INTERNATIONAL CO., LTD.
7F., NO. 110, Sec 3, Zhongshan Rd.
Zhonghe City, Taipei Country, 235,
Taiwan(R.O.C.)

EMERGO EUROPE
Molenstraat 15.
2513 BH The Hague.
The Netherlands
Tel+31(0)703 458 570
Fax+31(0)703 467 299

Issue Date:2010-05-01
Rev: 1

CE 0434